



ResMed

*Changing lives
with every breath*

Floton Auto CPAP Launch



14 March, 2019

➤ Floton Auto CPAP – Product Introduction

- ❑ Compact, Light and Quiet
- ❑ Automatic Positive Airway Pressure (APAP) treatment through most accurate pressure control which provides patient comfort
- ❑ Proprietary Expiratory Unloading Technique (EUT), equivalent to EPR, improves patient comfort by reducing expiratory pressure
- ❑ Floton Auto CPAP provides continuous positive airway pressure to the patient delivering optimal pressure breath by breath

➤ Floton Auto CPAP – Key Features

❑ EUT Exhalation Relief

The Floton Auto features EUT (Expiratory Unloading Technology) an exhalation relief function like many other popular machines. Exhalation relief can make therapy more comfortable and feel more natural. The EUT feature can be set on 1 to 3 different levels from a light amount of relief to a larger drop in pressure when exhaling. The EUT feature can also be set to 0 which turns the feature off entirely.

❑ Ramp Time

When using the ramp function the pressure starts at 4cmH₂O and rises until the device reaches the set PAP RAMP setting is between 0-60min and increase in pressure is 1 cm of water step per minute

❑ Advanced Data Recording

The Floton Auto will record up to 6 months of the following therapy information to the on-board memory. Data can be transferred through USB cable and viewed using the OPTIONAL AutoManager software. The parameters which get recorded are Apnea, Hypopnea, Snoring, Flow Limitation, Leak.

➤ Floton Auto CPAP – Key Features

❑ Power Out Alarm

If the power goes out while the Floton APAP machine is running, the machine sounds an alarm to notify the user of the disruption in power. To silence the alarm press the power key. The power out alarm will also sound if the power cord is disconnected from the wall outlet while the machine is running. The power out alarm function cannot be turned off.

❑ Auto On / OFF Feature

The Auto On / Off feature allows the Floton Auto to detect when the mask is on and the user is breathing. The machine automatically begins providing the therapy air stream without having to press a button. This feature may be turned off if desired.

If the AUTO ON feature is set to **OFF** and the mask is removed or there is a significant mask leak while the machine is blowing air, the mask off alarm will sound. The alarm can be silenced by pushing the up and down arrow keys and by turning the blower off.

➤ Other Features

- ❑ Mode – CPAP, Auto CPAP
- ❑ Pressure range – 4-20 cm H₂O
- ❑ Noise – < 30 dB
- ❑ Alarm – Power Failure, Mask off
- ❑ Weight – 1.2 Kg (1.6 Kg with Humidifier)



Thank You